



Advanced Communication Solutions Ltd
Units 21-23 Beaumont Close
Beaumont Industrial Estate
Banbury, OX16 1TG

Telephone: +44(0)1295 266 665

www.acscustom.com
www.hearingprotection.co.uk
info@acscustom.com

Instructions for the Audiologist: Ear impressions

To ensure the best fit of our custom products, ACS require a full shell impression, including the helix portion of the ear. The impressions must extend past the second bend of the ear canal.

We recommend the use of a high-viscosity silicone impression material, such as Dreve Otoform A Soft. For patient protection, the use of an oto-block is essential. We recommend foam blocks over cotton blocks.

The Impression material must be filled in fully to the oto block in the second bend and include every detail of the ear. The outer ear must be filled and extend out of the concha bowl, to collect the data of the Pinna. The picture below shows an ear impression detailing what we need to ensure the best fitting.

